


A17  
24/3

# NEW TEMPERATURE MEASURING BRACELET

PRODUCT DESIGN

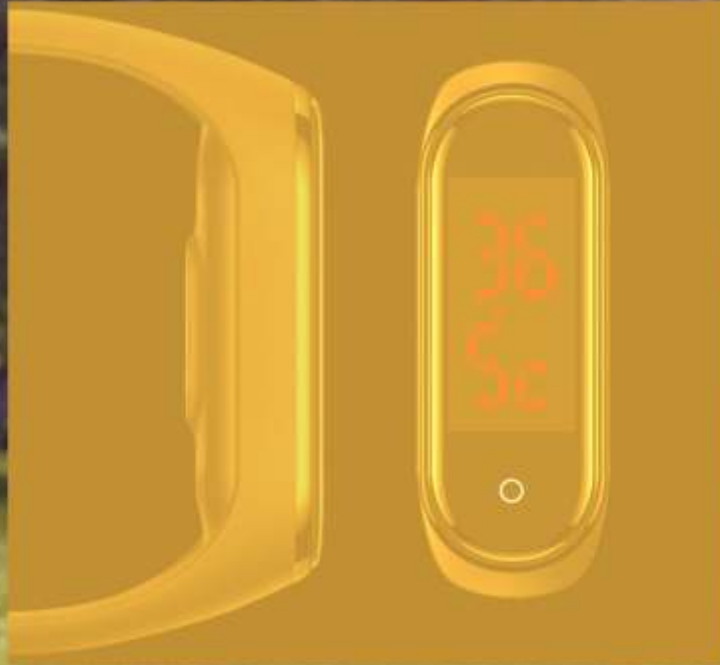

Time display



Test the temperature



## PRODUCT DESIGN

### Manual of Band

NEW TEMPERATURE  
MEASURING BRACELET

#### 1. Time display

Short press the button to display the time. it will be clocked in 5 seconds when there is no operation, automatically turns off to save electricity.

#### 2. Test the temperature

Under the state of time display, press the button to measure the temperature and the symbol °C flashes. test Stop flashing of symbol °C after test finished. There are two cases of abnormal temperature measurement: 1. LO or HI is displayed, indicating Temperature too low or too high, out of range. 2. High temperature warning: when the temperature is higher than 37.5° the red light on the top left corner of the LED screen flashes, warning.

Note: the temperature range is from 34°C to 42.5°C. Temperature measurements are influenced by individuals, environment, motion, Clothing and other factors, the results of temperature measurement for reference only, can not be used to determine the real temperature of the human body. Degree, can not be used for medical testing purposes, not as a judgment of the health of the body.

#### 3. Set the time

Under the state of time display, long press the button for 5 seconds to enter the setting of clock numeric value, and short press Modify the number of hours. Press the button again for 5 seconds to enter the minute value setting of the clock, short Press the button to modify the number of minutes.

#### 4. Charging

Take the inner core of the hand ring from the wrist band and connect it with the charging accessories. Insert the USB head of the charging accessories Enter the standard DC5V charger for charging. LED screen status display: charging center left top the corner red lights flickered. Full charge, red light on top left.

#### 5. Low power hint

When the battery is low, the bracelet will prompt the BALO symbol.



NEW TEMPERATURE  
MEASURING BRACELET

## PRODUCT DESIGN



### Monitoring temperature Focus on life

Wrist contact temperature monitoring, your health,  
temperature fluctuations, are accurate recognition.  
Record your body temperature and health, let  
you pay attention to every detail of life  
health, breathe naturally  
and relax every day.



Time display  
Accurate temperature  
measurement

A17



## PRODUCT DESIGN

Monitoring temperature  
Focus on life

Applicable population: children, students, teenagers, middle-aged adults, old people. Check the temperature at any time, master accurate information, and bring you and your family a healthy life.



NEW TEMPERATURE  
MEASURING BRACELET



Time display



Normal body temperature



Abnormal body temperature  
The icon in the upper left corner flashes



Charging in progress  
The icon in the upper left corner flashes

## MONITORING TEMPERATURE FOCUS ON LIFE

After the metal sheet on the back contacts the human skin, the human body temperature can be detected anytime and anywhere. In order to achieve the purpose of monitoring body temperature.





NEW TEMPERATURE  
MEASURING BRACELET



Monitoring temperature  
Focus on life  
Time display  
Accurate temperature  
measurement





Product parameters	
Manner of wearing	Wrist watch mode
CPU	High precision and low energy consumption microprocessor
Screen	LED display
Key	Touch key
Display time mode	24-hour display mode
Temperature measurement mode	Contact skin heat conduction
Temperature measurement range	30℃~45℃
Battery capacity	55mAh
Standby time	~90days (Sleep Mode)
Regular use time	25days (Temperature measurement and viewing time of normal human body)
Charging voltage	DC 5V
Charging mode	Contact charging
Size	57x20x12mm (Including watch strap)
Weight	17.7g
Colour	Black

■ ■ THANKS